

Artox Gel

Composition:

Product based on: water, ozonized olive oil, devil's claw glyceric extract, chondroitin sulphate, mountain arnica glyceric extract, camphor, panthenol, EDTA disodium salt, glucosamine sulphate, triethanolamine, menthol, glycerol, essential oil of peppermint, sodium chloride, sodium benzoate, carbomer, xanthan gum, phenoxyethanol, ethylhexylglycerin, potassium sorbate.

Indications:

Artox gel is a medical device indicated for localized trauma and contusions. Thanks to the osmotic action and its volatile components, the product is useful in giving relief from muscle or joint pain.

Dosage and method of administration:

Apply the product at least twice a day to the affected areas with a gentle massage to facilitate the absorption.

Storage conditions:

Store at room temperature. Store the product tightly closed and away from heat sources. The product should be used within 3 months from its opening.

Contraindications:

Do not use in case of hypersensitivity reactions to the components of the product.

Warnings:

The device is not subject to special precautions of use, we advise, however:

- Keep out of reach of children.
- Do not exceed the recommended doses.
- Do not use the product after the expiry date; the expiry date refers to the intact and properly stored product.
- Do not use the product if the packaging is not intact.
- After a short period of treatment (5 days) with no appreciable results, consult your doctor.
- In case of adverse reaction stop using the product and consult your doctor in order to establish an appropriate therapy.
- Compliance with the instructions contained in the leaflet reduces the risk of side effects. It is important to inform your doctor or pharmacist if you notice side effects not listed in this leaflet.

Interactions and incompatibilities:

The concomitant use of other products on the same area of application may cause the non-absorption of the components, the deterioration of their conditions or other unpredictable side effects.

Pregnancy and breastfeeding:

Pregnancy: there are no adequate data on the use of the product in pregnant women, therefore, in that case, it should not be used.


Breastfeeding: there are no adequate data on the use of the product in breastfeeding women, therefore, in that case, it should not be used.

Presentation:

Artox gel is a medical device and it is supplied in a 100 ml tube.



Gruppo Farmaimpresa srl, Via Cipro, 1-25124 Brescia (Italy)

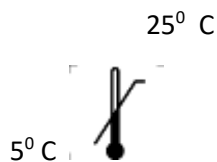

The medical device must not be disposed in the wastewater or household waste. Ask your pharmacist how to dispose of medical devices no longer required. This will help protect the environment.